



Funded by the European Union



CBRN
**Centres
of Excellence**
An initiative of the European Union



A EUROPEAN PROJECT CONTRIBUTION TO ADDRESS THE CORONAVIRUS EMERGENCY

ENTRANCE – DONNING

All operations are carried out in pairs with a colleague.

1) Remove all personal items (jewelry, watches, cell phones, pens, etc.)



2) Perform hand hygiene



3) While sitting, wear over-boots




4) Put on first pair of gloves (inner layer)



5) Wear the full suit helped by the colleague leaving the head uncovered
→ check suitability of the size



6) Wear a second pair of gloves (outer layer) helped by the colleague, adapting the edge of the glove over the sleeve of the suit
→ **check the integrity and comfort of the double gloves**

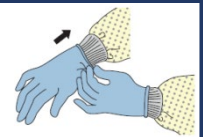

7) Put on the FFP3/FFP2 mask, arranging the rear elastic bands, adapting the profile on the face, around the nose and checking the good seal



8) Wear protective goggles, adjusting the rear elastic bands, and check the position of the glasses around the eyes



9) Put on the hood checking the rim cover of the glasses
→ for female personnel: carefully collect long hair (if any) inside



10) Check the comfort of the suit (with a bend on the knees)